JAY Pelvic Positioning Belts

EN

Important Consumer Information.

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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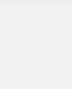
	
ENGLISH	4
WARNINGS / INSTALLATION	4
WARRANTY	5

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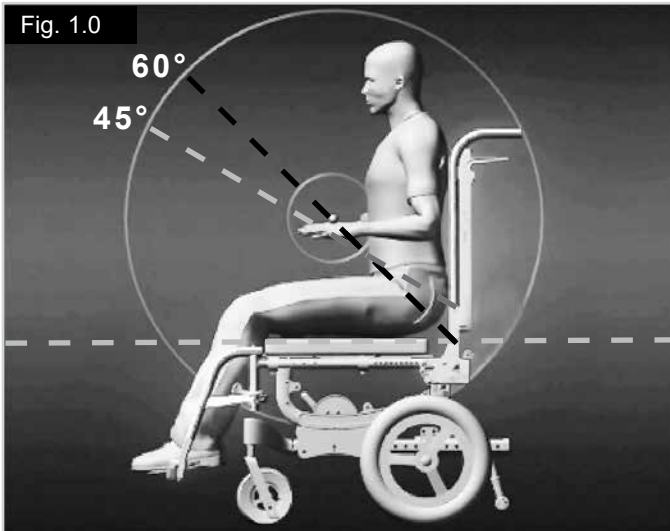
As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

SHIPPING:

Check all parts are free from shipping damage. In case of damage DO NOT use. Contact Sunrise Medical or your supplier for further instructions.

WARNING!

- JAY Positioning belts were designed for positioning, not to be used as transportation safety belt!
- JAY POSITIONING BELTS may be used while in transit but are not crash tested or approved to provide protection during a crash.
- An appropriate clinical assessment by a clinician or qualified wheelchair professional is highly recommended to guarantee safety and correct clinical usage.
- Any belt must not interfere with breathing.
- Never place a pelvic belt, or positioning support across the waist or abdomen.

The products described in this manual were designed to be fitted and installed exclusively by:

- An authorized Sunrise Medical supplier.
- A qualified and competent healthcare professional who has undergone appropriate certified training by either Sunrise Medical or their approved supplier.

PELVIC BELT INSTALLATION:

Initial Considerations:

The angle that the pelvic belt is mounted to the wheelchair has a direct impact on the pull angle on the pelvis (Fig. 1).

A 60° attachment angle to the seat rail positions the pelvic belt anterior and inferior to the axis, which reduces the tendency of the user with a posterior pelvic tilt to slide under the belt.

A 45° attachment angle on the back post encourages a posterior pelvic tilt, which assists in positioning users with an anterior pelvic tilt.

Pelvic belts can be installed in different ways according to user's seating system, wheelchair type and hardware used (Fig. 2-3).

All JAY POSITIONING BELTS include belt anchors for attachment (Fig. 4).

SAFETY:

For both types of belt, when the installation is finished, carry out the following procedure to ensure that the user is appropriately and safely positioned:

1. Check if the user is sitting comfortably.
2. Check if there are any areas of irritation. This usually manifests as a reddening of the skin especially in soft tissue.
3. Check that the user does not slide down through the belt, if they do, adjust the belt as required. If the user still slides under the belt and the belt is a two point model, assess if there is a need for the four point belt, this will provide additional security and adjustment.
4. Check if the pelvic belt mounting hardware is interfering with any other positioning accessories/ devices on the wheelchair and reposition it if necessary.

24 MONTH LIMITED WARRANTY

Each Product is carefully inspected and tested to provide peak performance. Every Product is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to any removable cover.

Removable covers are guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorised supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S