

Vacuvet® 23 B+

Quality - you can rely on.



Decades of experience in conjunction with the specialized expertise in infection control guarantee our most reliable solution for steam sterilization: Vacuvet® 23 B+ combines durable and sturdy technology with ease-of-use. Discover the product highlights aligned to the requirements in veterinary clinics:



Are you looking for a safe and reliable autoclave with a long life-span? Follow the example of 500,000 practices and clinics throughout the world and take advantage of the superior MELAG quality - made in Germany.



Short cycle times and effective vacuum drying ensure the safe sterilization of all your instruments. Our aligned accessories make for optimal loading of the autoclave with up to 5 kg of instruments.



Vacuvet® 23 B+ monitors all relevant process parameters using a highly precise microprocessor control to ensure highest safety. The program logs provide legal certainty and are saved via CF card, network connection or printer.



One autoclave – Two systems: With integrated water storage containers and an efficient air-cooling, the stand-alone concept make the autoclave completely independent of location and easy to install. The use of an additional MELAG water treatment unit ensures a fully automatic and time-saving water supply.

Technical data

Figures and facts at a glance

Programs	Quick-Program S*	Quick-Program B**	Universal-Program	Gentle-Program	Prion-Program
Program specification	134 °C, Holding time: 3,5 Min.	134 °C, Holding time: 5,5 Min.	134 °C, Holding time: 5,5 Min.	121 °C, Holding time: 20,5 Min.	134 °C, Holding time: 20,5 Min.
Sterilization time	15 min.	29 min.	31 min.	45 min.	45 min.
Drying time	5 min.	10 min.	20 min.	20 min.	20 min.

*For unwrapped instruments only

**Max. load 1.5 kg wrapped or 5 kg unwrapped

Device type	Chamber size	Chamber volume*	Load quantity	Device dimensions (W x H x D)	Weight	Power supply	Power consumption
Stand-alone	45 cm long x Ø 25 cm	23 litres	5 kg instruments 1.8 kg textiles	42,5 x 49 x 74 cm Device fits on a 60 cm table top	50 kg	230 V / 50/60 Hz	2.100 Watts

*Litre specification rounded

i The steam sterilizer is conceived for use in a veterinary context only. It is not a medical device as defined by 93/42/EEC or EU 2017/745 and thus has not been approved for use in a human medical context.



For the safety of our best friends:

Do you have any questions about the correct instrument decontamination procedure required in a veterinary clinic? Simply contact our Hygiene Experts in your country: The MELAG partners are keen resolving individual questions in your clinic or via an online Live-Demo.

For further information, please visit: www.melag.com

MELAG
competence in hygiene