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Instructions for use

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Thank you for choosing our product. NOEL has designed and manufactured Positioning Bath Bagel Stabilo System to the highest safety and functional use standards. We are confident that our product will meet your highest requirements and expectations. These instructions for use are an integral part of Positioning Bath Bagel Stabilo System. Please read the instructions thoroughly before using the product.

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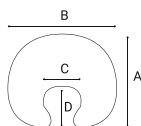
Vacuum positioning system stabilizing head Bath Bagel Stabilo System

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Vacuum positioning system Bath Bagel Stabilo System is a product designed for children and adults with cerebral palsy, compensating for and alleviating the disability by positioning the head. It can be used to stabilize and protect the head when bathing.

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In the present instructions, the term cushion is used to refer to the Positioning Bath Bagel Stabilo System.

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PRODUCT SAFETY WARNINGS

- Use of airy cover is advisable if increased ventilation is required around the body
- Avoid contact with sharp objects that may damage the cushion
- Do not clean using solvent-based cleaners, e.g. toluene or acetone
- Do not pull on valve
- Do not overinflate cushion
- Use only special pump included with cushion for inflation
- Do not iron
- In case of tear, micro-granules may escape which may be dangerous for small children

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Due to the risks stemming from Positioning Bath Bagel Stabilo System usage (the possibility of choking on water when taking a bath) additional safety measures apply.

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- Never leave the user unattended.
- **Remember! Bath Bagel will not prevent drowning. It is only used to stabilise the head.**
- Remain doubly vigilant.
- Bath Bagel should be put on before entering the bath. Ensure the head rest is stable, so that when a user tilts their head back they will not choke on water
- Before each use, check adherence of straps to the soft Bath Bagel outer layer

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ATTENTION

- The thickness of the cushion can naturally reduce by 5% in the initial period of use
- In case of considerable changes of temperature use pump again to obtain proper hardness (higher temperature softens the cushion, lower temperature hardens the cushion)

INSTRUCTIONS FOR USE

Hand pump:

1. Take out tube in pump handle.
2. Screw tube into cushion valve and wait until cushion reaches the required softness.
3. Shape cushion (either manually to adjust shape or by sitting or leaning on it).
4. Screw pump into tube and deflate air until desired hardness has been reached.
5. Remove tube from pump and screw protecting cap on valve.
6. To change shape - screw tube into valve and wait until cushion softens, then repeat steps from 2 to 5.

Electric pump

1. Take out tube from box with electric pump.
2. Screw off valve cap, screw tube (tapped part) into cushion valve, and push second tube into SOFT socket in electric pump. Turn on pump and wait until cushion will be soft enough, then turn pump off.
3. Start shaping cushion (either manually to adjust shape or by sitting or leaning on it).
4. Change position of tube from SOFT to HARD and turn pump on removing air until the desired hardness has been reached.
5. Screw off tube, put cap back on and tighten.
6. To change the shape repeat steps from 2 to 5.

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
ATTACHING

The attachment method is significant to ensure maximum user safety.

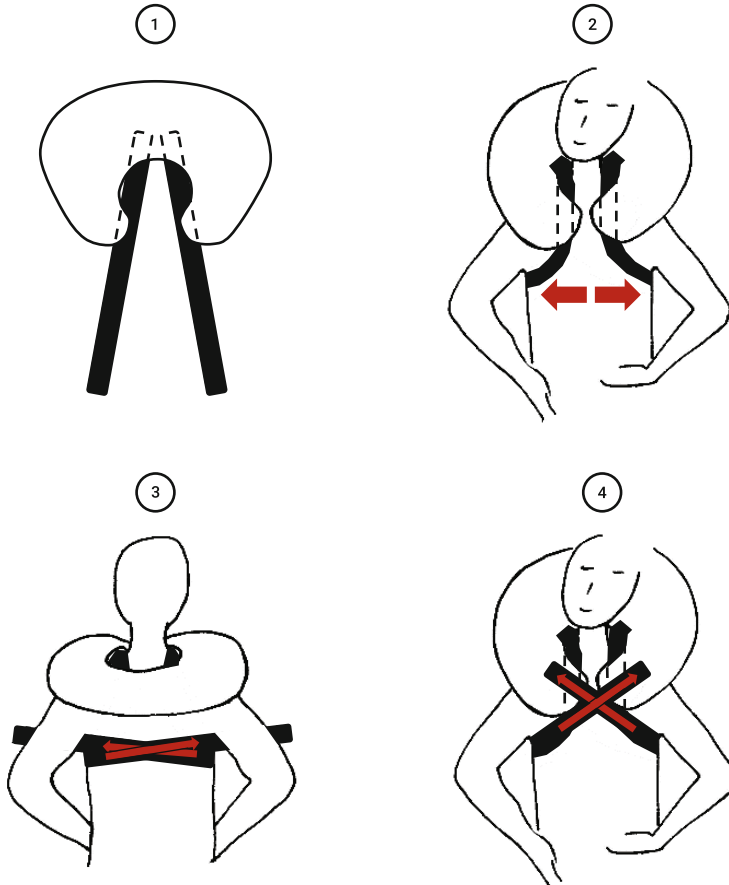1. Attach straps to the inside of the neck opening (see fig. 1).
2. Pass straps under arm pits (see fig. 2).
3. Cross the straps on the back, then pass them again under arm pits (see fig. 3).
4. Attach straps diagonally, to the opposite end of the cushion, near the chin (see fig. 4).
5. The straps should be tensioned so that the cushion is attached to the body and not the head.

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MAINTENANCE

- Regularly check condition of product – pay attention to any cracks and rips
- Screw on valve cap before washing
- Hand wash cushion under running water using a standard detergent, e.g. a washing up liquid
- Suitable for washing in washing machine at 40°C cycle – fabric softener is sufficient, does not require use of washing powder
- Wipe dry with paper towel
- Standard disinfectants used in hospitals can be used
- Do not use solvent-based cleaners such as toluene or acetone
- For Velcro products, Hoover the soft surface regularly to prevent loss of adherence.

WARRANTY

- The warranty is provided for a period of 24 months
- The warranty does not cover mechanical damage to Positioning Bath Bagel Stabilo System
- If in doubt, please contact seller or manufacturer directly
- Report any incidents to seller or manufacturer directly

DECLARATION OF CONFORMITY

Positioning Bath Bagel Stabilo System medical device complies with the basic requirements of the Directive 93/42/EEC.

NOEL s.c.
ul. Romanowicza 6, 30-702 Kraków
+48 12 341 53 16
www.stabilo.krakow.pl
stabilo@stabilo.krakow.pl

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