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Instructions for use

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Positioning Mattress  
Stabilo System

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Thank you for choosing our product. NOEL has designed and manufactured Positioning Mattress Stabilo System to the highest safety and functional use standards. We are confident that our product will meet your highest requirements and expectations. These instructions for use are an integral part of Positioning Mattress Stabilo System. Please read the instructions thoroughly before using the product.

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### Vacuum positioning system stabilizing torso, head and limbs Positioning Mattress Stabilo System

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Vacuum positioning system Mattress Stabilo System is a product designed for children and adults with cerebral palsy and myelomeningocele compensating for and alluviating the disability by stabilising patient in sitting or lying position. Mattress Stabilo System is suitable for people with large deformations of the body, making it difficult to lie down or sit, with risk of developing pressure sores.

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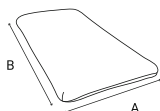
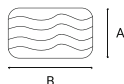
#### PRODUCT SAFETY WARNINGS

- For Grande do not lie down on a layer of granulate which is too thin
- Use of airy cover is advisable if increased ventilation is required around the body
- Avoid contact with sharp objects that may damage the mattress.
- Do not clean using solvent-based cleaners, e.g. toluene or acetone
- Do not pull on valve
- Do not overinflate mattress

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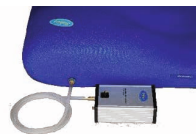
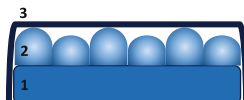
- Use only special pump included with mattress for inflation Do not iron
- In case of tear, micro-granules may escape which may be dangerous for small children
- Only persons who received instructions or training may use the device
- Do not use Grande Armadilo near flammable materials (water) or open flame (air pumped into the mattress supports a flame)
- Keep Grande Armadilo pump away from other electronic devices, however if proximity to other electronic devices is unavoidable, make sure that it and the other devices operate correctly
- Change the user's position every 5 hours

## ATTENTION

- The thickness of the mattress can naturally reduce by 5% in the initial period of use
- In case of considerable changes of temperature use pump again to obtain proper hardness (higher temperature softens the mattress, lower temperature hardens the mattress)

## INSTRUCTIONS FOR USE ELECTRIC PUMP

1. Take out tube from box with electric pump.
2. Screw off valve cap, screw tube (tapped part) into cushion valve, and push second tube into SOFT socket in electric pump. Turn on pump and wait until cushion will be soft enough, then turn pump off.
3. Start shaping cushion (either manually to adjust shape or by sitting or leaning on it).
4. Change position of tube from SOFT to HARD and turn pump on removing air until the desired hardness has been reached.
5. Screw off tube, put cap back on and tighten.
6. To change the shape repeat steps from 2 to 5.

## ADDITIONAL INFORMATION FOR GRANDE ARMADILLO

Grande Armadilo consists of

1. Grande mattress
2. Pressure-relief insert
3. Membrane cover

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## USE

1. Shape the mattress to suit to user's needs.
2. Put the pressure-relief insert on the bed, on the Stabilo Grande (the mattress sockets are to be near the users legs and facing upwards).
3. Pull the elastics under the mattress, and click the clips.
4. Place the pump on a flat surface or hang it on a wall or bed. Connect the pump to the air sockets, check quality and firmness of connections.
5. Set the green switch to ON position. The mattress will be ready for use after 20 minutes; once pumped up, adjust the pressure in the insert's bubbles using the knob, according to the user requirements SOFT/HARD.
6. Spread the cover over the insert (the matte surface upwards, the membrane downwards). Lay down the user on the mattress.

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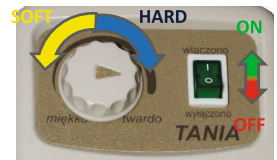
## MAINTENANCE

- Regularly check condition of product – pay attention to any cracks and rips
- Screw on valve cap before washing
- Hand wash mattress under running water using a standard detergent, e.g. a washing up liquid
- Suitable for washing in washing machine at 40°C cycle – fabric softener is sufficient, does not require use of washing powder
- Wipe dry with paper towel
- Standard disinfectants used in hospitals can be used
- Do not use solvent-based cleaners such as toluene or acetone

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**WARRANTY**

- The warranty is provided for a period of 24 months
- The warranty does not cover mechanical damage to Positioning Mattress Stabilo System
- If in doubt, please contact seller or manufacturer directly
- Report any incidents to seller or manufacturer directly

**DECLARATION OF CONFORMITY**

Positioning Mattress Stabilo System medical device complies with the basic requirements of the Directive 93/42/EEC.

NOEL s.c.  
ul. Romanowicza 6, 30-702 Kraków  
+48 12 341 53 16  
[www.stabilo.krakow.pl](http://www.stabilo.krakow.pl)  
[stabilo@stabilo.krakow.pl](mailto:stabilo@stabilo.krakow.pl)

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