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Instructions for use

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Thank you for choosing our product. NOEL has designed and manufactured Headrest Velcro Stabulo System to the highest safety and functional use standards. We are confident that our product will meet your highest requirements and expectations. These instructions for use are an integral part of Headrest Velcro Stabulo System. Please read the instructions thoroughly before using the product.

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## Vacuum positioning system stabilizing torso, head and limbs Headrest Velcro Stabulo System

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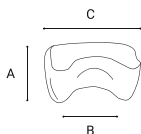
Vacuum Positioning Headrest Velcro Stabulo System is a product designed for children and adults with cerebral palsy compensating for and alleviating the disability by stabilizing and positioning head. Cushion Headrest Velcro Stabulo System with the possibility of asymmetrical modeling can be used for patients with torso distortions and for flaccid patient.

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Headrest Velcro Stabulo System cushions are covered a special outer material that allows attaching velcro strips.

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### PRODUCT SAFETY WARNINGS

- Positioning Headrest Velcro Stabulo System must be securely fixed to seats or backrests
- Use of airy cover is advisable if increased ventilation is required around the body
- Avoid contact with sharp objects that may damage the cushion
- Do not clean using solvent-based cleaners, e.g. toluene or acetone
- Do not pull on valve
- Do not overinflate cushion
- Use only special pump included with cushion for inflation
- Do not iron
- In case of tear, micro-granules may escape which may be dangerous for small children
- Only persons who received instructions or training may use the device

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## ATTENTION

- The thickness of the cushion can naturally reduce by 5% in the initial period of use
- In case of considerable changes of temperature use pump again to obtain proper hardness (higher temperature softens the cushion, lower temperature hardens the cushion)

## INSTRUCTIONS FOR USE

Hand pump:

1. Take out tube in pump handle.
2. Screw tube into cushion valve and wait until cushion reaches the required softness.
3. Shape cushion (either manually to adjust shape or by sitting or leaning on it).
4. Screw pump into tube and deflate air until desired hardness has been reached.
5. Remove tube from pump and screw protecting cap on valve.
6. To change shape - screw tube into valve and wait until cushion softens, then repeat steps from 2 to 5.

Electric pump

1. Take out tube from box with electric pump.
2. Screw off valve cap, screw tube (tapped part) into cushion valve, and push second tube into SOFT socket in electric pump. Turn on pump and wait until cushion will be soft enough, then turn pump off.
3. Start shaping cushion (either manually to adjust shape or by sitting or leaning on it).
4. Change position of tube from SOFT to HARD and turn pump on removing air until the desired hardness has been reached.
5. Screw off tube, put cap back on and tighten.
6. To change the shape repeat steps from 2 to 5.

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## MAINTENANCE

- Regularly check condition of product – pay attention to any cracks and rips
- Screw on valve cap before washing
- Hand wash cushion under running water using a standard detergent, e.g. a washing up liquid
- Suitable for washing in washing machine at 40°C cycle – fabric softener is sufficient, does not require use of washing powder
- Wipe dry with paper towel
- Standard disinfectants used in hospitals can be used
- Do not use solvent-based cleaners such as toluene or acetone
- Hoover the soft surface Velcro regularly to prevent loss of adherence

## WARRANTY

- The warranty is provided for a period of 24 months
- The warranty does not cover mechanical damage to Headrest Velcro Stabilo System
- If in doubt, please contact seller or manufacturer directly
- Report any incidents to seller or manufacturer directly

## DECLARATION OF CONFORMITY

Positioning Headrest Velcro Stabilo System medical device complies with the basic requirements of the Directive 93/42/EEC.

NOEL s.c.  
ul. Romanowicza 6, 30-702 Kraków  
+48 12 341 53 16  
www.stabilo.krakow.pl  
stabilo@stabilo.krakow.pl

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