## HUDSON RCI® TRIFLO® II

<b>EN</b>	<b>INCENTIVE DEEP BREATHING EXERCISER INSTRUCTIONS FOR USE</b>	<b>ES</b>	<b>ESAKİTME KİMLİĞİNİN BAĞLARI ANAHTAR ODAĞIÇI EKİPMAN</b>
<b>DE</b>	<b>GERBRAUCHSANWEISUNG</b>	<b>CS</b>	<b>CVČENÍČEK POMOČKA PRO HLUBOKÉ DYCHANÍ</b>
<b>FR</b>	<b>DISPOSITIF D'ENTRAÎNEMENT INCITATIF À LA RESPIRATION PROFONDE</b>	<b>IT</b>	<b>STIMULATORE DI CAVITÀENZA GEROK-IEGO ODDYCHANIA INSTRUKCJA OBSŁUGI</b>
<b>NO</b>	<b>DIPEE ADEMTRAINER</b>	<b>SK</b>	<b>SPROBUIVAČNÍK TRÉNÁŽER ZA GLOBOKO DIHANJE</b>
<b>PT</b>	<b>SPRIMOTIVO INCENTIVANTE PER GINNASTICA POLMONARE CON INSPIRAZIONE PROFONDA</b>	<b>TR</b>	<b>NAVODILA ZA UPORABO</b>
<b>RU</b>	<b>ИНСТРУКЦИЯ ПЕР ЛЬУЮ</b>	<b>UK</b>	<b>АПАРАТ ДІЄКТИВ ПЕНТУ-ІНСПІРАЦІЯ</b>
<b>ES</b>	<b>INSTRUKCJA DLA PACJENTA</b>	<b>HR</b>	<b>INSTRUKCIJA UPOTREBA</b>
<b>IT</b>	<b>ISTRUCCIONES DE USO</b>	<b>SE</b>	<b>STIMULERINGSPRIMOTIV</b>
<b>FR</b>	<b>RESPARACIÓ PROFUNDA TRIFLO® II</b>	<b>PL</b>	<b>LEŻEŚGWAKOŁKI KĘSZÓŁEK</b>
<b>DE</b>	<b>INSTRUKCJE DLA UŻYTKOWNIKA</b>	<b>NO</b>	<b>LEZÉSZAKLÁNY TÁJASZTÁS</b>
<b>PT</b>	<b>STIMULANSPIROMETER</b>	<b>HR</b>	<b>LEŽEŠKALNÝ POSILOVAČ NA HLBOKÉ DYCHANJE</b>
<b>ES</b>	<b>INSTRUKTIONER FÖR ANVÄNDNING</b>	<b>SK</b>	<b>NAVOD NA POUŽITIE</b>
<b>NO</b>	<b>KANNUSTAVA SYVÄÄNHENGITYSHARJOTUSLAITE</b>	<b>TR</b>	<b>TEŞVİK EDİCİ DERİN NEFES GİRİŞİZİ ALTI</b>
<b>DE</b>	<b>MOTIVERENDE TRAININGSAPPARAT FÜR DYBERTRÄKNING</b>	<b>UK</b>	<b>КУЛІНАННІ ТАЛМАТЛАРИ</b>
<b>RU</b>	<b>INSENTVPIROMETER</b>		
	<b>BRUKSANVISING</b>		

<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>
			

- Remove the components from the package and attach the open end of the tubing to the stem in front of the exerciser. With the unit in an upright position, exhale normally and place your lips tightly around the mouth-piece.
- Exhale. Inhale at a sufficient rate to raise only the ball in the first chamber, while the ball in the second chamber remains at rest. The position should be held as long as possible. A hold maneuver should then be maintained for three seconds.
- For a Higher Flow Rate: Inhale at a rate sufficient to raise the first and second balls, while the ball in the third chamber remains at rest. Exhale. Inhale at a sufficient rate to raise only the ball in the first chamber, while the ball in the second chamber remains at rest. After performing exercise, remove the mouthpiece from your lips and exhale normally. Relax. Following each prolonged deep breath, take a moment to rest and breathe normally. Then repeat exercise as directed by your physician.

### How to clean your TRIFLO® II Incentive Deep Breathing Exerciser

- Cleaning instructions:**
- Remove the mouthpiece from the tubing.
  - Fill a ratio of 1 ml of soft detergent into 1 L of warm tap water in a clean tub or bowl.
  - Immerse the mouthpiece in the soapy solution and soak for approximately 5 minutes.
  - After soaking but while the component is immersed, wipe the interior and exterior of the mouthpiece with a non-linting wipe for a minimum of 30 seconds.Rinse the mouthpiece under warm running water for a minimum of 1 minute or until all soap is removed.
  - Dry the mouthpiece using non-linting wipes.
  - Visually inspect the mouthpiece to ensure it is clean. Repeat cleaning steps if residual soil is present.
  - Reconnect the mouthpiece to the tubing.

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## en TRIFLO® II INCENTIVE DEEP BREATHING EXERCISER

### Slow Sustained Maximal Inspiration

Helps develop, improve, and maintain your respiratory fitness
Your TRIFLO® II Incentive Deep Breathing Exerciser has been scientifically constructed as a means of encouraging you to take a slow Sustained Maximal Inspiration (SMI). A Sustained Maximal Inspiration is vital important to your general well-being. Deep breaths expand the small air sacs of your lungs and help clear the air passages of mucus and other irritants.
Normally you take many deep breaths each hour, usually without being aware of it. They are spontaneous and automatic, and occur in the form of sighs and yawns. In certain instances, however, your normal breathing pattern can change. When you are experiencing pain following chest or abdominal surgery, for example, breathing tends to become shallow, and deep breaths are suppressed in an effort to minimize pain.
In these instances, it is important that you re-learn your normal breathing pattern despite any discomfort you may have. Taking the deep breaths that you might ordinarily suppress will help prevent the possibility of respiratory complications. By carefully following your physician's instructions and the directions provided with your TRIFLO® II Incentive Deep Breathing Exerciser, you should be well on your way toward better breathing.

**Intended use:**
An Incentive Spirometer is a device that indicates a patient's breathing volume or flow and that provides an incentive for the patient to improve his or her ventilation.

**Indications of use:**
The Hudson RCI® Triflo® II Incentive Deep Breathing Exerciser is intended as an inspiratory deep breathing positive exercise for adult and pediatric (above 5yr's) patients. Intended for single patient, multi-use in a hospital or home care setting. This product is intended to be used for a maximum of 150 uses/cycles. Accuracy of spirometer: +/- 400 cc/seg.

**How to use your TRIFLO® II Incentive Deep Breathing Exerciser**

1. Remove the components from the package and attach the open end of the tubing to the stem in front of the exerciser. With the unit in an upright position, exhale normally and place your lips tightly around the mouth-piece.

2. Exhale. Inhale at a sufficient rate to raise only the ball in the first chamber, while the ball in the second chamber remains at rest. The position should be held as long as possible. A hold maneuver should then be maintained for three seconds.

3. For a Higher Flow Rate: Inhale at a rate sufficient to raise the first and second balls, while the ball in the third chamber remains at rest. Exhale. Inhale at a sufficient rate to raise only the ball in the first chamber, while the ball in the second chamber remains at rest. After performing exercise, remove the mouthpiece from your lips and exhale normally. Relax. Following each prolonged deep breath, take a moment to rest and breathe normally. Then repeat exercise as directed by your physician.

**How to clean your TRIFLO® II Incentive Deep Breathing Exerciser**

**Cleaning instructions:**

- Remove the mouthpiece from the tubing.
- Fill a ratio of 1 ml of soft detergent into 1 L of warm tap water in a clean tub or bowl.
- Immerse the mouthpiece in the soapy solution and soak for approximately 5 minutes.
- After soaking but while the component is immersed, wipe the interior and exterior of the mouthpiece with a non-linting wipe for a minimum of 30 seconds.Rinse the mouthpiece under warm running water for a minimum of 1 minute or until all soap is removed.
- Dry the mouthpiece using non-linting wipes.
- Visually inspect the mouthpiece to ensure it is clean. Repeat cleaning steps if residual soil is present.
- Reconnect the mouthpiece to the tubing.

**Caution/Warnings:**

- Single patient use: the use of products intended for single patient use on more than one patient creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single patient use only may result in degraded performance or a loss of functionality.
- DO NOT use if the device is damaged.
- If a serious incident has occurred in relation to the device, it should be reported to Medline and the competent authority of the Member State in which the user and/or patient is established.

**Distributed in Australia by:** Medline International Two Australia Pty. Ltd., Sydney, Australia

**Medline Industries, LP,** Three Lakes Drive, Northfield, IL, 60093 USA  
 Medline International France SAS, 5 rue Charles Lindbergh, 44110 Châteaubriant, France

**Distributed in Australia by:** Medline International Two Australia Pty. Ltd., Sydney, Australia

**UK representative and importer in UK:** Medline Industries Ltd, WA4 6H LUK

**CH Rep and importer:** Medline International Switzerland SARL, c/o M/N & Associés SA, 1 Place de Longeville, 1204 Genève, Switzerland

**Medline International BV,** Nieuwe Statustrasstraat 10, Arnhem, 6811 KS, Netherlands

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